de WICHTIGE GEBRAUCHSINFORMATION (Winkel-Prothese Plester)

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en IMPORTANT MEDICAL INFORMATION (Angular Prosthesis Plester)

- Protheses for Tympanoplasty

DESIGN	PRODUCT/ TYPE	REFERENCE NUMBER	MATERIAL	WEIGHT (mg)	HEIGHT OF BELL (mm)	LENGHT (mm)	INDICATIONS FOR USE	IMPLANT AND INTENDED SITUS
	Angular Prosthesis Plester	1002 610 1002 612	Titanium (ASTM F67 Medical Grade)	3	1	2.25 3.25	For bridging defects at the long incudal process and otherwise intact mobile ossicular-chain	The prosthesis consists of a slit bell, an angular stem, and two clamps at its end. The open ends of the clamps are slightly staggered, so that they do not touch each other. The bell is placed on the stapes head. In case of a missing stapes head, it is placed on the still intact arch of the stapes crura. The clamps hold the stump of the long incudal process, thus keeping the prosthesis in position.

Information for Use Protheses for Tympanoplasty (Angular Prosthesis Plester)

Apply this Information for Use within the context of the “General Information for Ossiculoplasty Protheses”.

Surgical Technique:	The surgical techniques are demonstrated at various surgical seminars. Call the manufacturer/distributor for details or check programs of professional conferences. Attention: Removal from Package: Be absolutely sure to avoid accidental bending of implant during withdrawal from plastic container; otherwise proper function of the protheses might be impeded.
Sterility:	KURZ middle ear protheses are provided sterile. Packaging contents are sterile, provided that the packaging was not damaged or opened. KURZ protheses are sterilized with gamma radiation in strictly controlled cycles. Open package directly before surgery to prevent contamination and observe the usual instructions for asepsis upon withdrawal.
Resterilization/ Reprocessing:	The prosthesis is intended for one-time use only. Reprocessing / resterilization is not permitted.
Warning:	The physician must inform the patient about the following: Patients with metallic implants are not to be subjected to microwave radiation. Severe variations in ambient pressure (scuba diving, diving head-first, explosions, fireworks, etc.) can damage the middle ear structures and/or cause hearing loss and/or equilibrium disorder and should be avoided.
MR Safety Information:	MR Conditional at 1.5 T; 3.0 T and 7.0 T, for detailed Information please refer to the MR Information on www.kurzmed.com .
Liability:	Complications or other adverse affects which might occur due to incorrect indication or unsatisfactory surgical technique, selection of unsuitable handling of them, asepsis, etc. come under the liability of the surgeon.
Storage:	Storage conditions in unopened original packaging: Store in a dry place at room temperature, protected from direct sunlight. Brief fluctuations of temperature and humidity are permissible. Each prosthesis bears a batch number and an expiration date and may not be used after that date.
Sale Restriction:	US federal law restricts KURZ protheses to sale by or on the order of a physician.
Documentation:	It is recommended by the manufacturer to record the LOT-Number and Type of Prosthesis in the patient's medical record, operative record and implant pass by applying the adhesive labels.
Manufacturer:	Heinz Kurz GmbH, Tuebinger Strasse 3, 72144 Dusslingen, Germany; Phone: +49 (0)7072 / 9179 - 0; Fax: +49 (0)7072 / 9179 - 79; Internet: www.kurzmed.de ; E-mail: info@kurzmed.de USA: Kurz Medical Inc., 70 Chestnut Street Shrewsbury, MA 01545-USA. Phone: 508.841.5900 Fax: 508.519.2672 Toll Free: 866.449.8020 Customer Service: orders@kurzmed.com

SYMBOLS

	Manufacturer		Item number
	Expiration date		Batch code
	Sterilized using irradiation		Do not use if package is damaged
	Keep away from sunlight		Do not resterilize
	Do not reuse!		Single sterile barrier system
	Store in a dry place		Single sterile barrier system with protective packaging inside
	Consult instructions for use		Single sterile barrier system with protective packaging outside
	Fragile, handle with care		Medical device
			Caution

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