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USA GB IMPORTANT INFORMATION / KURZ PROTHESES FOR TYMPANOPLASTY

MRP – Malleus Replacement Prosthesis (Designed by R. Vincent)
Read carefully before clinical use!

Apply this Information for Use within the context of the “General Information for Ossiculoplasty Prostheses”.

Product Description

KURZ middle ear prostheses are intended for ossicular replacement to restore functionality to the middle ear in cases of pathological changes of the middle ear. The MRP prosthesis helps to stabilize KURZ partial and total prostheses.

DRAWING	NAME/ TYPE	REF	SIZE	MATERIAL	PURPOSE	PROPOSED SITE
	MRP Malleus Replacement Prosthesis (Designed by R. Vincent)	1006 960	0.8 x 3.0 mm 	Titanium (ASTM F67)	The MRP prosthesis is specially designed to stabilize a KURZ partial or total prosthesis on the head plate, through which the ossicle reconstruction is performed.	

Application

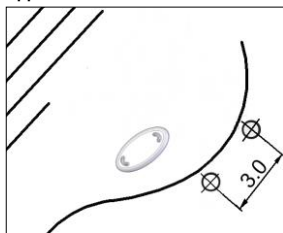
Figure 1:
Drill the two holes inside the auditory canal wall. Positioning of the holes should take account of the anatomical bone situation. Drill hole diameter = 0.6 mm, depth approx. 2.0 mm, center-to-center distance 3.0 mm. Flushing with water should be performed constantly throughout the drilling procedure.

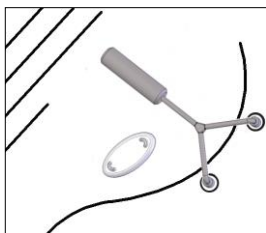
Figure 2
Place the two pistons of the MRP into the holes.

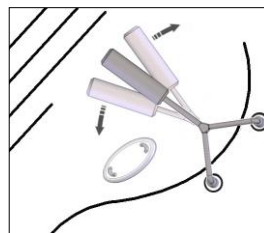
Figure 3
Adjust the MRP to the anatomical geometry by carefully bending the shaft.

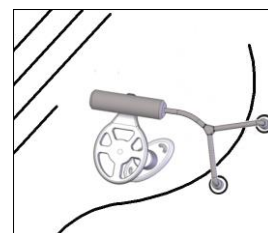
Figure 4
Placement of a partial or total prosthesis. Stabilization with the MRP. Then cover the footplate, incl. the MRP, with transplanted tissue.

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